







View images:

On desktops / laptops:

 Left button drag	Performs the currently selected interaction function. The default mode is Zoom .
 Right button drag	Window / Level
 Both buttons drag	Pan
 Middle button drag	Fast Scroll
 Wheel scroll	Advance through the images at full quality
 Right click:	Displays the context menu

On mobile devices (iPhone, iPad, Droid, Blackberry, etc)








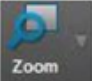




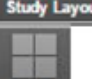

 Touch drag	Performs currently selected interaction function. The default mode is Scroll .
 2 fingers	Pinch zoom and pan
 3 fingers	Window / Level
 Tap and hold	Context menu
 Shake	Reset

Image Tool Bar Functions:

Image Layout:

Scroll	 Scroll	2D Drag to scroll forwards or backwards through the images in a series.
Pan	 Pan	Drag to move an image within a viewport.
Zoom	 Zoom	Drag up to zoom in on an image , and drag down to zoom out.
Rotate	 Rotate	Rotate image clockwise or counter clockwise
Invert	 Invert	Invert grey images. Will be applied to all images in the series.
Window Level	 Window Level	Drag to adjust the window level.
Reset	 Reset	Sets image back to original state when first opened.
Study Layout	 Study Layout	Use to display multiple series on screen
View	 View	Use to select the layout for a specific series (1x2, 2x2, etc)

Lexmark Nil Quick User Guide



Perform measurement:

- Select desired measurement from Annotations drop-down tool on toolbar



- Click/touch and drag for rulers, ellipses, ROIs etc.,

Keyboard Shortcuts:

a	Axial Orientation		
c	Coronal Orientation		
e	Start / Pause Cine		
Page Down	Next Page / Image		
Page Up	Previous Page /		

Help:

- Slide toolbar to right . click Help